iMask-Cradle - User's Manual (CHN)

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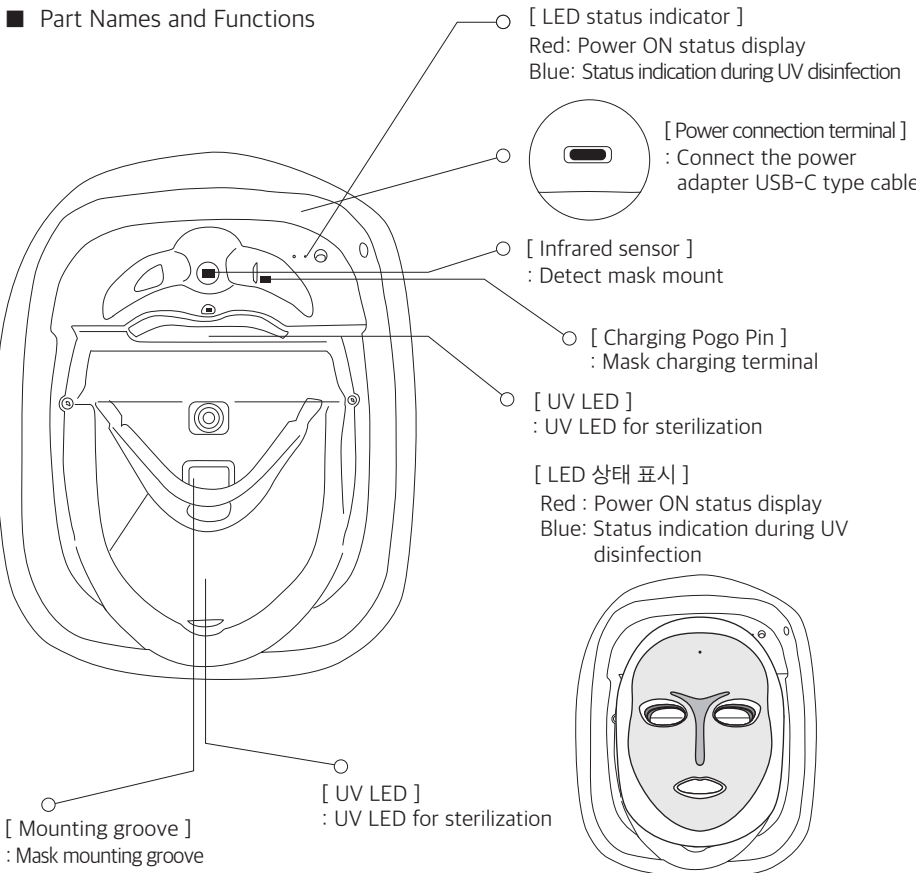
■ Warranty

- This product is manufactured under strict product quality compliance and examination process of Miinsystems co., ltd.
- Repair is free of charge only when malfunction happens under users' ordinary use(Within Warranty Period)
- Repair needs to be paid, if the type of breakdown is out of Free A/S service.

Product Name	miin-iMask	Warranty Period	1 year from the purchase
Model Name	Artemis-C		
Serial No.			
Date of Sales			
Customer's Address			
Customer's Name		Tel	

- Guide for Free repair
In case the product is broken within 12 months from the purchase, free repair would be provided when it has been used in ordinary use environment.
- Guide for Paid repair
Repair needs to be paid in the following cases
 - Malfunction caused by user's mistake and mishandling failure.
 - Out of warranty period.
 - Malfunction caused by customer's intention or mistake(carelessness).
 - Malfunction and damage caused by other service engineer's repair.
- Product consultation and A/S guide : www.miinsystems.com/ 031-385-9969

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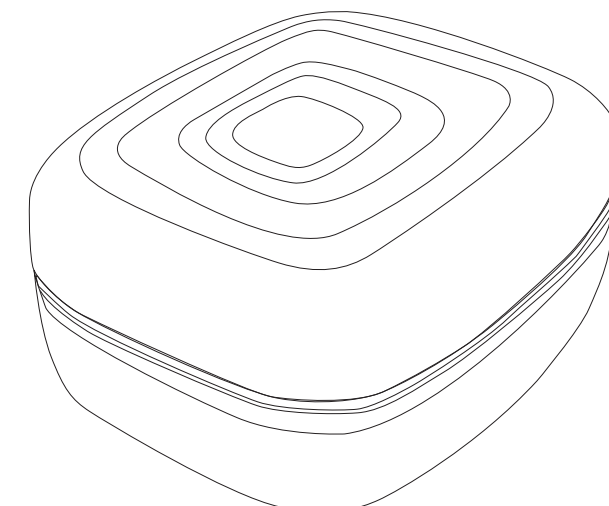
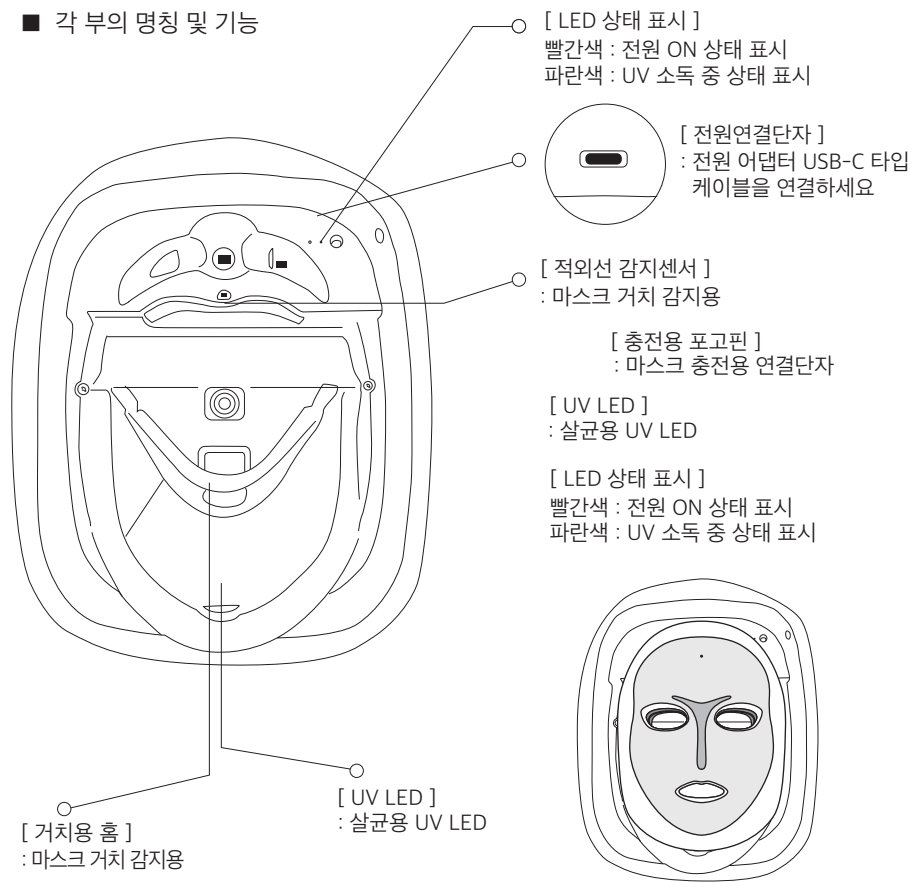
■ Warnings

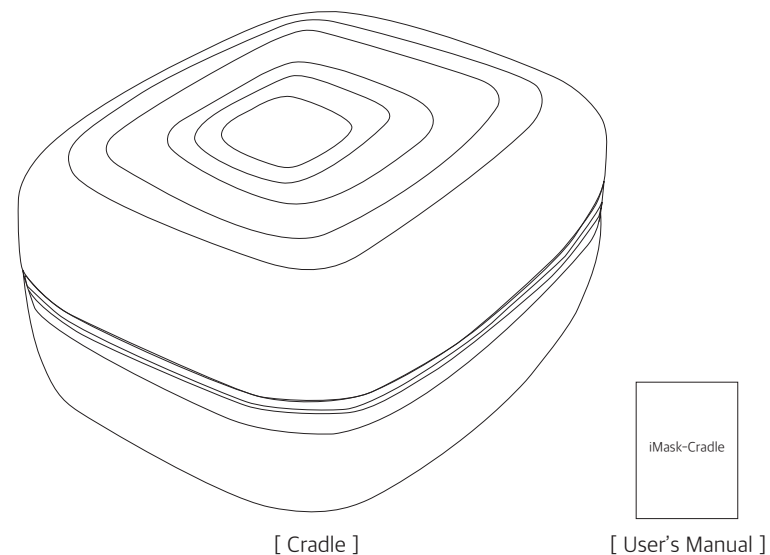
If you do not follow the instructions, it may cause equipment damage and personal injury.

- Do NOT put the device in the heater or microwave range. It may cause device damage or explosion.
- Do NOT use or charge when it gets wet. It may become the cause of a fire, leaks, excessive heat or rupture.
- NEVER turn it ON when it gets wet. It may cause malfunction.
- Do NOT disassemble or give the device a shock. It may be damaged if you drop it on the hard floor.
- Keep your face or hands from the device when it is broken. It may cause your skin injury.
- Please put in the power plug tightly. Do NOT bend, twist, heat or cut the power cord. It can be broken and cause a fire or electronic shock.
- Do not look into the LED while the device is in operation. Your eyesight may be impaired.
- Do not remove the product while the device is in operation.

■ Power Supply

- Connect the USB-C type power connector to the power connector.
- At this time, the red LED turns on.

iMask-Cradle - User's Manual (ENG)

■ Charging the battery of Mask

- When the mask is placed on the cradle, charging is automatically performed.
- When fully discharged, it takes about 3 hours to fully charge.
- * CAUTION : Place the mask correctly on the cradle.

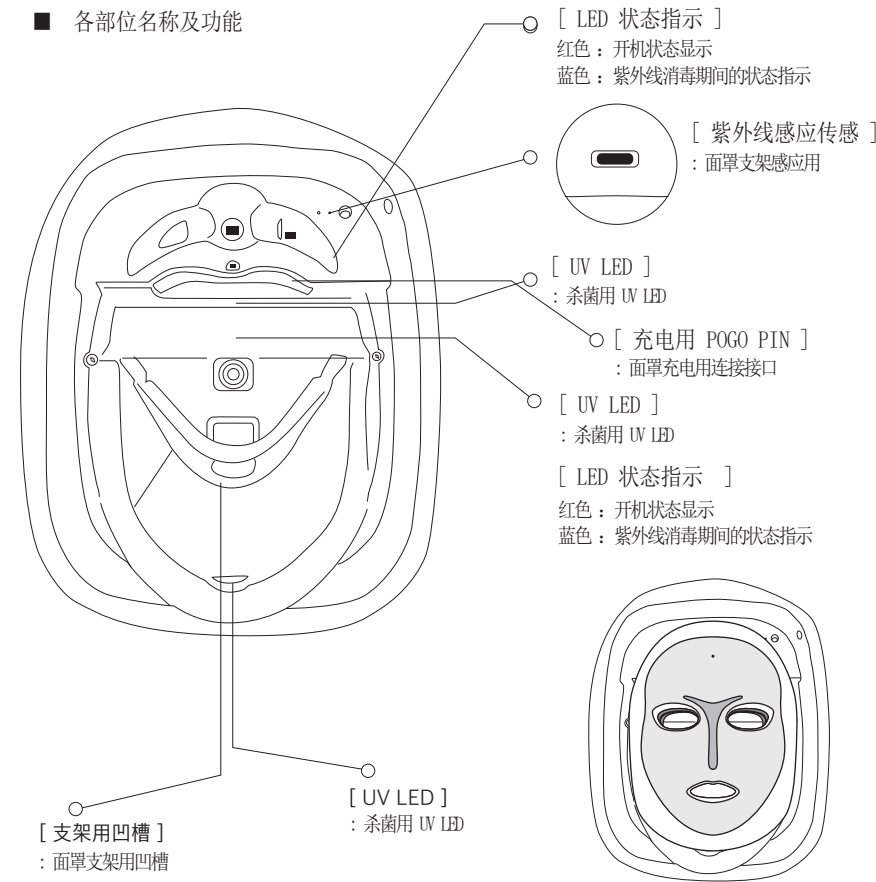
■ Sterilization

- When the mask is placed on the holder, the sensor senses and sterilization proceeds automatically.
- Stop after 30 minutes of sterilization
- * CAUTION : Do not look directly at the UV LED with your eyes.

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